

the **PortaSCC**[®] milk test

Detect elevated somatic cell counts while the infection is sub-clinical **BEFORE** clinical symptoms appear.

Easy to use!

- Collect sample: quarter, composite or bulk tank.
- Dispense one drop of sample onto test strip well using a pipet from kit.
- Flush well with three drops of Activator Solution, included in kit.
- Allow 45 minutes for color development.
- Read result using convenient Quick Check Color Chart or optional Reader.

PortaSCC Test Kits

Includes: 12 two-test strip packs, color chart, 1 blanking strip, 5 reusable transfer pipets & 1 bottle of activator solution.

24 tests/kit

Stock# 218-3024



Includes: 50 two-test strip packs, color chart, 2 blanking strips, 20 reusable transfer pipets & 1 bottle of activator solution.

100 tests/kit

Stock# 218-3100



PortaSCC Digital Strip Reader

Includes: 1 digital reader with 2-year battery

Optional Reader

Stock# 218-3001



PortaCheck
A Division of PortaScience Inc.
www.portacheck.com

**Trials available...
Call for details!**

DISTRIBUTED BY:
 **Farm Friend Direct**
A SERVICE OF NELSON-JAMESON, INC.
www.farmfrienddirect.com

CALL 800-342-3877
FAX 800-472-0840

New Product Announcement

PortaSCC® Somatic Cell Count Test *from* Nelson-Jameson

This test permits detection of elevated somatic cell counts while the infection is sub clinical – before clinical symptoms appear. And before the CMT or stripping reveal a problem.

Test cows at freshening or dry off and take appropriate measures early during these critical times for effective treatments. Use with on farm culture plates to identify causes of high SCC counts when infections are most susceptible to treatment, then during remedial therapies to monitor results.

The test is very easy to perform using strips and accessories included in the kit.

- 1) Collect the sample: quarter, composite, or bulk tank.
- 2) Dispense one drop of sample onto test strip well using a pipet from kit.
- 3) Flush well with 3 drops of Activator Solution, included with kit.
- 4) Allow 45 minutes for color development.
- 5) Read result using convenient Quick Check Color Chart or optional Reader.

To order or obtain a free set of kit components for a trial at your location, call Nelson-Jameson, Inc. When ordering, please refer to the following item numbers and descriptions.

Test Kits:

218-3024 PortaSCC Test Kit, 24tests/kit
*(includes: twelve 2-test strip packs, color chart,
1 blanking strip, 5 reusable transfer pipets &
1 bottle of activator solution.)*

218-3100 PortaSCC Test Kit, 100tests/kit
*(includes: fifty 2-test strip packs, color chart,
2 blanking strips, 20 reusable transfer pipets &
1 bottle of activator solution)*

Optional Reader:

218-3001 PortaSCC Digital Strip Reader
(includes: 1 digital reader (SCC Meter) with 2-year battery)

More information about PortaSCC may be found online at: www.portacheck.com



Now serving you from three locations:

**Marshfield, Wisconsin
Turlock, California
Twin Falls, Idaho**

E-mail – sales@nelsonjameson.com ▪ www.nelsonjameson.com
Phone 800-826-8302 ▪ Fax 800-472-0840